


Special Applications

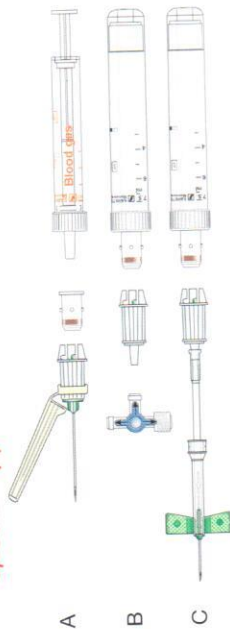

- The membrane adapter (A) can be used if, in exceptional cases, blood is to be collected with a Luer-Monovette® (e.g. blood gas).
- The S-Monovette® can be used for blood collection from Luer connections (3-way-tap, Butterfly, etc.) by means of the multi adapter (B).
- For difficult vein conditions, we recommend to use the Safety-Multity® (C) with integral multi adapter.

User Guide S-Monovette® Serum/Serum Gel

Make sure to observe the following instructions for optimal serum yield after blood collection using the S-Monovette® Serum:



After blood collection:
Store S-Monovette® in an upright position for 30 minutes



During the clotting phase (i.e. the initial 30 minutes after blood collection), it is essential to store the S-Monovette® in an upright position to ensure a distinct separating layer after centrifugation and to avoid irregular effects (i.e. 'sausage' pattern)!